Ruthinium  
**DISC**  
easy

PMMA CAD-CAM DISC

---

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 Fabbricante

**(GB) PMMA DISK FOR CAD-CAM PROCESSING. DISC MADE FROM CROSS-LINKED HIGH MOLECULAR WEIGHT RESIN WHICH ENSURES A FAITHFUL REPRODUCTION OF DETAILS AND RESISTANCE EVEN TO THIN THICKNESSES.**

**SCOPE OF APPLICATION:** Used for the manufacture of temporary anatomic bridges and/or crowns, intended to stay in the mouth for a maximum of 30 days. Can also be used to make castable pieces.

**CONTRAINDICATIONS:**

- In case of proven allergy to components of Ruthinium Disc Easy.
- All uses not indicated.

**AVAILABLE FORMATS:** ATTACK WIEBER (Ø 98,5 mm) thicknesses: 16mm – 20mm

**COLOURS AVAILABLE:** A2 – A3

**MAIN CHARACTERISTICS:**

- Medical device for CAD/CAM dentistry: the dental laboratory uses a fine milling manufacturing process to obtain temporary dentures, and modelled structures such as crowns and temporary bridges.
- High stability.
- Chromatic inalterability.

**INSTRUCTIONS FOR USE:**

1. Processing of the material must take place in an environment under controlled conditions so that the material is not contaminated. In case of contamination from grease or oils, clean the material with alcohol.
2. Vacuum the chips during machining, avoid overheating of the material to avoid deformation.
3. Finish as usual, with burs, rubber points, brushes and with pumice. Mirror polish with the help of brushes and polishing products (Acry Lux Crema, Acry Lux Liquido, Acry Lux Pasta).

**WARNINGS:**

- Sterilise the device before use in the oral cavity.
- Ruthinium Disc Easy can be sterilised with ethylene oxide or by ionising radiation.
- The product is not autoclavable.
- Avoid steam cleaning.
- Any non-eugenol cement available on the market can be used for cementing.

**STORAGE CONDITIONS:** It is recommended to store the product in a cool and dry place. Ruthinium Disc Easy is not a dangerous product. Disposal must take place in accordance with the laws of competent authorities. Waste is treated as urban waste or recyclable as normal plastic.

The product is designed for professional use in the dental sector. Follow the instructions for use. The manufacturer shall accept no responsibility for damages arising from different or inadequate use. The user must personally check the suitability of the product for the intended use, particularly if these uses are not reported in the instructions for use. Do not use after expiration.

Warning: Any serious accident that may occur with the device must be reported by the user to the manufacturer and the competent authority of the Member State where the user and/or patient are established.

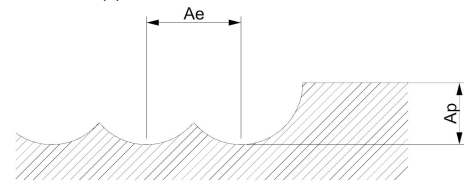
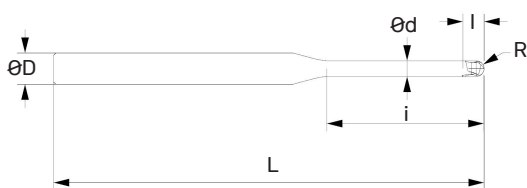
Instruct the patient with adequate cleaning protocols.

**RECOMMENDED WORKING STRATEGY**

TOOL TYPE							MILLING PARAMETERS				
Ø TOOL mm	Ø d mm	Nr. of Blades	R mm	l mm	i mm	L mm	RPM (S)	TOOL FEED XY (F)	TOOL FEED Z (F)	Ae (XY)	Ap (Z)
2	2	2	1	1.6	20	50	18500	1200	800	1	0.5
1	1	2	0.5	0.8	16	50	25000	1000	600	0.15	0.08
0,6	0,6	2	0.3	0,5	10	50	30000	500	500	0.05	0.05

ØD = Constrained by gripper measure of the milling machine

TOOL TYPE and MILLING PARAMETERS = recommended values and working strategy, consult the tool supplier.

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**Ruthinium® - Dental Manufacturing S.p.A.**